

BIYOVENT

Intensive Care Type
Mechanical Ventilator Device





BREATHE THE FUTURE

BiYOVENT

Intensive Care Type

Mechanical Ventilator Device





OUR BUSINESS: SOURCE OF THE LIFE

We produce technology for breathing, and breathing for health.

As Biosys, we offer solutions for the deficiencies in the medical sector with competent engineers and specialist doctors having many years of R&D experience since 2012. In our working process, we are progressing with the awareness that benefiting from health services is the most fundamental social right and that this right must be provided to every individual and we develop our technologies with this awareness.

We make our ideas a reality in our R&D center in order to meet both national and global needs in the field of health by developing new generation medical technologies. We know that even the smallest work and business we carry out can affect the health of an individual on earth, and for this, we carry out meticulous and deep studies to design every detail in our products perfectly. Thanks to our effective and safe production approach, we produce new generation medical technologies at high quality and reliability, and we hope for a healthy life.

In the technologies we have developed for the needs and demands of the health sector, we do not only advance and improve by relying on our engineering expertise, but also we carry out a multidisciplinary study shaped by the opinions of doctors in every step we take. Thus, we make a difference in the health sector with medical devices that have high technological threshold values and fully meet the needs, and have the best design, software and technology. With this difference, we continue to progress confidently to affect the health of every individual and to be the gateway to a healthy life in the world.

THE NEXT STEP IN VENTILATOR TECHNOLOGY



Biyuvent, which was developed with the aim of offering products with high technological threshold value, fully meets the needs with its new generation technology. It offers high quality in terms of both technology and health, thanks to its features that cover all the deficits that the ventilator devices used up to this time cannot offer and the doctors cannot meet the demands.

With every detail designated from design to hardware, from software to technology and developed with a holistic approach, Biyuvent adds value to health with its modern and ergonomic design, user-friendly interface, traditional and innovative operating modes, and adaptable structure to different patient groups. In

the ventilators used until this time, the fact that there are more parts that change according to the needs of the patient and these parts are very expensive in terms of cost are minimized with Biyuvent.

This situation, which causes economic and technical disadvantages in ventilator devices, is no longer a disadvantage with the new generation ventilator technology developed for the need. With its ability to meet the needs of many patients by itself and the need for minimum extra parts, Biyuvent adds an innovative understanding to ventilator device technology and provides an advantage with its economic structure.

DESCRIPTION OF PERSONALIZED TREATMENT

Effective Treatment For the Needs of Every Patient

The medical state and needs of every patient are different. Biyovent which is developed with this awareness, offers a customizable structure based on patient needs. Thus, it provides advantages to both the patient and the doctor for the most effective treatment.

With Biyovent, in which every detail is considered for health, characteristics such as height and weight of the patients are entered during use in pediatric and adult patients, and the most appropriate treatment method is provided to the patient.

In addition, the changing clinical needs can be followed up by recording the values related to the patient's condition besides

physical characteristics. Thus, it is ensured that the most appropriate and correct treatment can be performed to each patient.

In terms of usage, Biyovent helps healthcare professionals to reach information they need and provides the opportunity to choose Non-Invasive, Invasive Ventilation and N-CPap, HFO therapy methods in performing the treatment method that the patient needs. With its advanced software features, Biyovent, which establishes a safe infrastructure for treatment, maximizes safety with the ability to select the length and thickness of the air canal circuit to be used for the patient from the air canal settings and adjustable alarm limits.



Invasive and Non-Invasive Ventilation



Integrated Nebulizer



High Flow Oxygen Therapy and N-CPAP (nasal C-PAP)



**Suitable for Neonatal (Optional)
Pediatric and Adult Patients**




Smart Ventilation Modes



biyovent

49.6 30.4 100.0 0.0 0.0

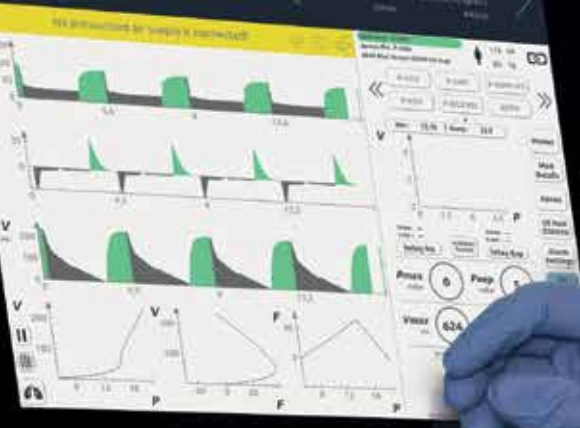

INNOVATIVE DESIGN, COMPREHENSIVE SOLUTIONS

Design Suitable for Quick and Easy Installation

Biyuvent is installed quickly and easily with its triple structure. Thanks to its ergonomic design consisting of foot, support and main body parts connected to the screen, it provides ease of installation. Biyuvent, which allows parts to be

connected with each other with eight screws, helps to operate quickly in a solution-oriented manner thanks to its simplified installation system rather than a complex configuration during device installation.



- 15 inch high definition color screen
- Sensitive touch
- Easy to switch between modes
- Easily adjustable mode parameters
- Simultaneous monitoring of more than 30 parameters.

- Power supply for humidifier
- External Nebulizer Input (USB)
- Emergency valve



- Multiple-use autoclavable Expiratory block
- Nebulizer outflow



- Wheels with increased shock absorption
- Lock mechanism
- Safe design
- Easy to use



ADVANCED SOFTWARE, EASY TO USE

With its easy-to-use interface design, patient details can be entered easily and the device provides the opportunity to customize parameters according to the patient.



Allows easy calibration in 3 steps with directions on the screen.



User-Driven Display System

With its advanced software features and user-driven interface design, Biyovent simultaneously displays 3 graphics, 3 cycles and lung dynamic status. Thanks to the visual aided interface design used in accessing information, it provides convenience in learning and practice, allowing access to more than 30 parameters on the same screen. With its high definition of 15 inches, it shows the information on the system clearly and in detail. Adjustments are made easily on the screen,

which has sensitive touch feature, and quick steps are taken for the necessary adjustments in the appropriate treatment for the patient. In addition to being able to monitor the instantaneous measurement values by displaying graphically for sudden situations related to the patient, it also helps to get quick intervention by giving warnings in case of alarm. In addition, it allows to see the dynamic state of the lung with the lung diagram, which offers intelligent ventilation monitoring.



It allows to monitor/ see the dynamic state of lung with lung diagram, which provides intelligent ventilation monitoring.




Monitoring parameters can be accessed on a single screen, allowing more than 30 parameters to be monitored simultaneously.





**APPROPRIATE
TREATMENT FOR
EVERY PATIENT**

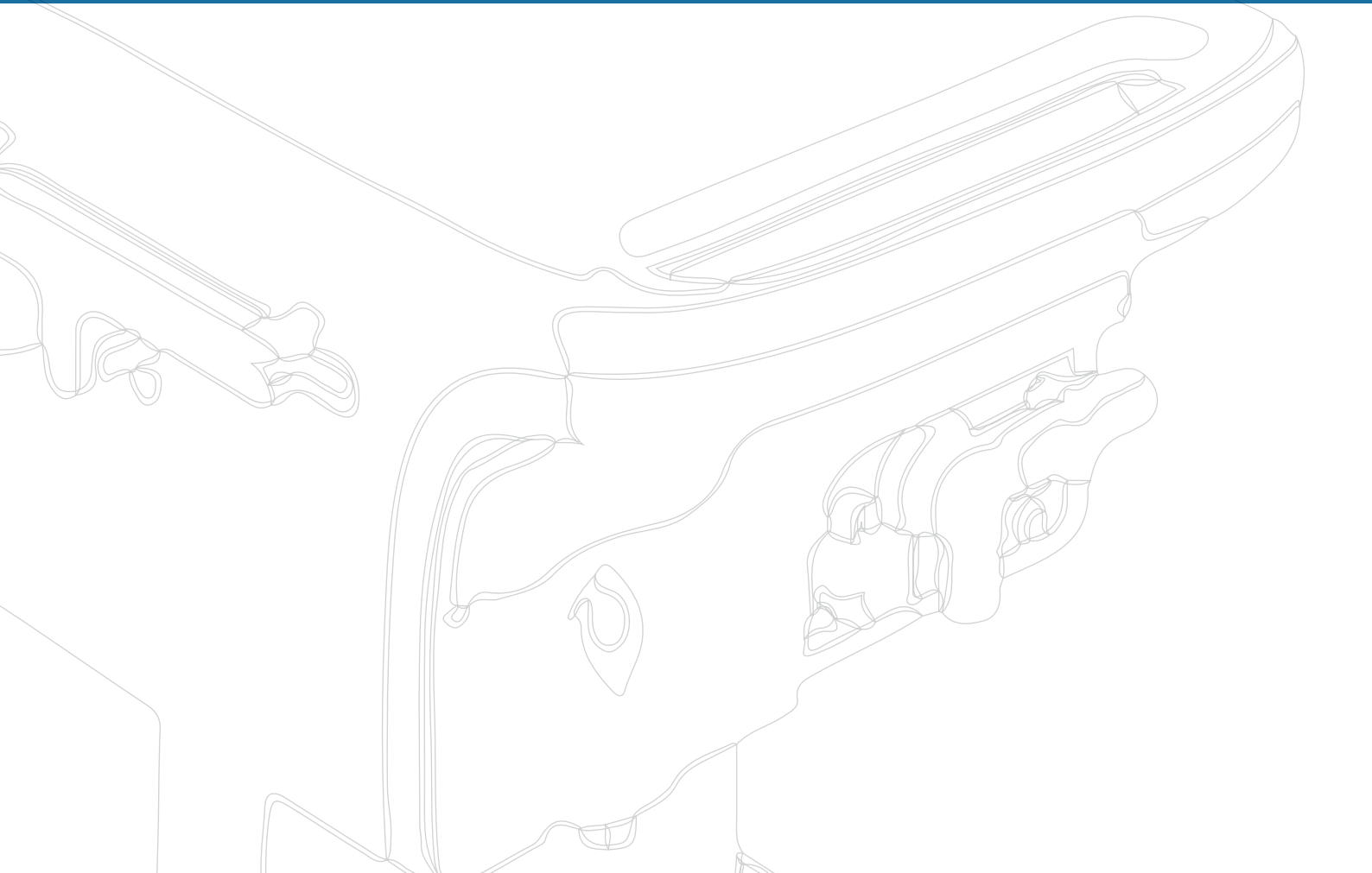



Ventilation Modes for All Ages and Conditions

- ▶ Pressure Controlled Modes (P-ACV P-SIMV+PS P-CMV P-PSV P-Bi-level APRV)
- ▶ Volume Controlled Modes (V-ACV V-ACV(PRVC) V-CMV V-SIMV+PS, V-SIMV(PRVC)+PS)
- ▶ Spontaneous Modes (SPN-PS SPN-VS)
- ▶ Smart Modes [SAV(WOB Optimisation)]



BIYOVENT

Intensive Care Type Mechanical Ventilator Device

TECHNICAL SPECIFICATIONS

Pressure-Targeted Modes
P-ACV
P-SIMV+PS
P-CMV
P-PSV
P-Bilevel
APRV
Volume-Targeted Modes
V-ACV
V-ACV(PRVC)
V-CMV
V-SIMV+PS
V-SIMV(PRVC)+P
Spontaneous Modes
SPN-PS
SPN-VS
Optional Features
Use in neonatal patients
SAV (WOB Optimisation)

Monitoring Parameters	
Ppeak (cmH2O)	Compliance (mL/cmH2O)
Peep (cmH2O)	RC Constant (sec.)
Minute Volume (Lt/min.)	Leakage (%)
Tidal Volume (ml)	WOB (joule)
Breath Number (Number/min.)	WOB/Lt (joule/Lt)
Pplato (cmH2O)	Vresidual (mL)
Port (cmH2O)	Vexpiratory (mL)
I:E Rate	Vinspiration (mL)
Ti (sec.)	AutoPEEP (cmH2O)
Te (sec.)	P0.1 (cmH2O/ 0.1s)
Inspiration Peak Flow (Lt/min.)	RSBI (ss/min)Lt
Expiratory Peak Flow (Lt/min.)	PTP (cmH2Oxsec)
Oxygen (%)	P_nif (cmH2O)
Minute Volume (Spn) (Lt/min.)	Expiratory Volume/min. (Lt/min.)
Breath Number (Spon) (Number/min.)	Leakage Constant (Ltr/m)/cmH2O
R Airway (cmH2O/ Ltr/sec)	Instant Tidal Volume (mL)
Elestance (cmH2O/Ltr) Elestance (cmH2O/Ltr)	rcExp (sec.)

Patient Suitability and Performance Suitability Child, Paediatric

Breath Number: 1-150/mins.

T inspiration: 0.1-10 sec.

Tidal Volume: 0.02-0.3 Litre

Flow: 1-60 Litre/Mins.

Adult

Breath Number: 1-100/mins.

T inspiration: 0.1-10 sec.

Tidal Volume: 0.1-3 Litre

Flow: 2-120 Litre/Mins.

Operation Features

Inspiration Pressure: 2-100 mBar

Inspiration Duration: 0.1-10 Sec.

Peep Pressure: 0-50 mBar

Breathe Speed: (ç) 1-150/mins. (y) 1-100/mins.

Flow: (ç) 1-60 lt/mins. (y) 2-120 lt/mins.

O2 Mixture: 21-100%

Spontaneous Pressure Support: 0-100 mBar

I/E Rate: 1:10 (x60*)-10:1 (ç): Child, (y): Adult

Pressure Welding Features

O2 Pressure: 2.5 - 7 Bar Central System/Tube

Air Pressure: 2.5 - 7 Bar Central System/Tube

Automatic change and alarm display at the end of welding

Operating with Medical compressor / Regulator

Electrical Features

Battery Duration: 2 hours + 8 hours (Optional)

Mains Voltage: 180 - 264 VAC

Power Consumption: 47 - 63 Hz 100W

Size and Weight

H: 150 cm / D: 58 cm / W: 58 cm

Weight: 55 Kg

Display movement: To the right or left 150° to up and down 15°

Display: Full touch screen


Digital Interfaces

USB: 4 pcs. / COM: 2 pcs.

HDMI: 1 pcs. / LAN: 2 pcs.

Display Port: 1 pc.

Mic. In: 1 pc.

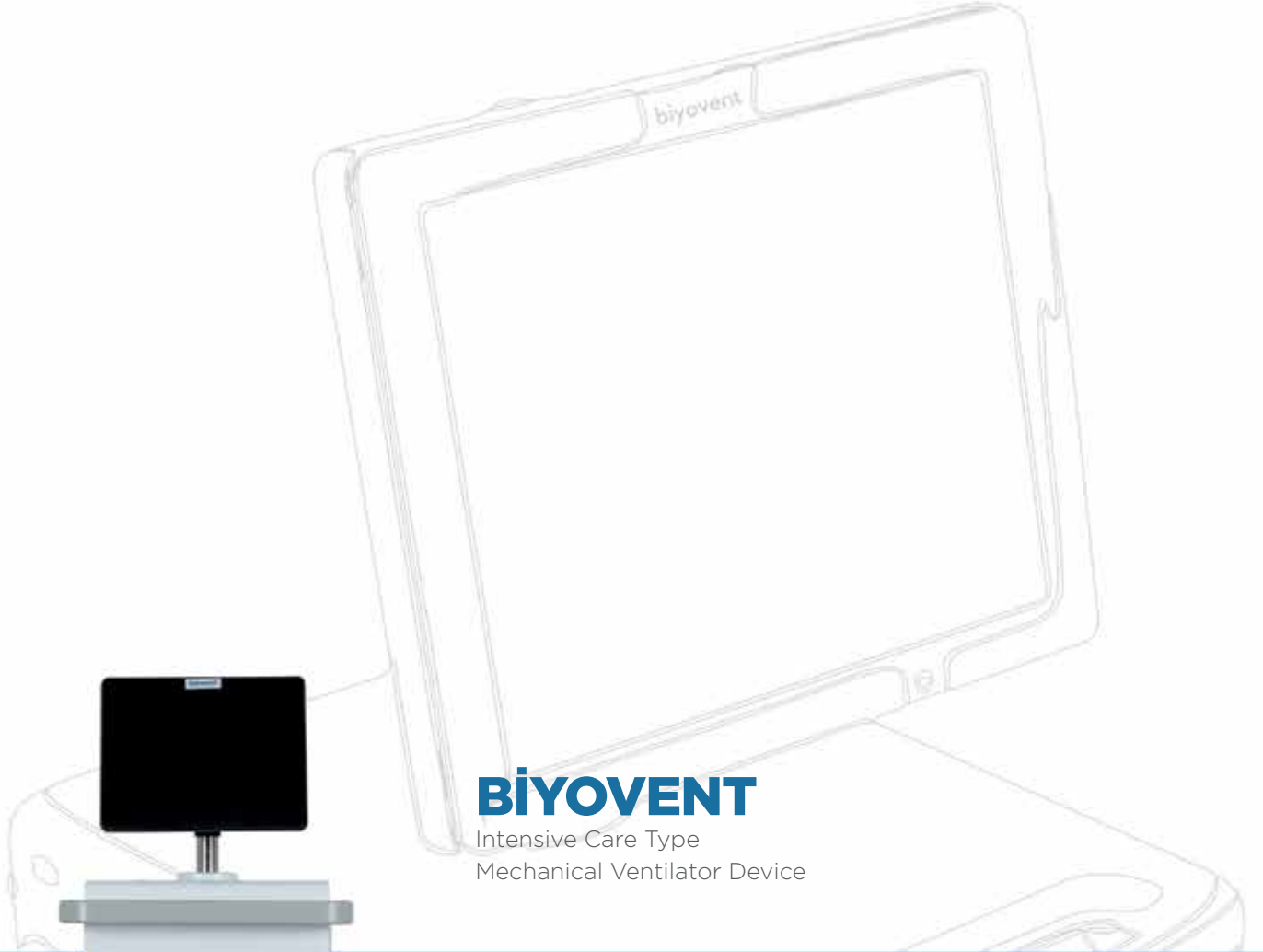

BİYOVENT

Intensive Care Type
Mechanical Ventilator Device

TD.15-AC/1020/Rev01

Üniversiteler Mahallesi İhsan Doğramacı Bulvarı No: 23/C
ODTÜ Teknokent 06800 Çankaya/ANKARA

0850 800 62 97

www.biosys.com.tr



All rights reserved. Technical specifications shall only be changed by Biosys Biomedical Engineering Ltd. Şti.